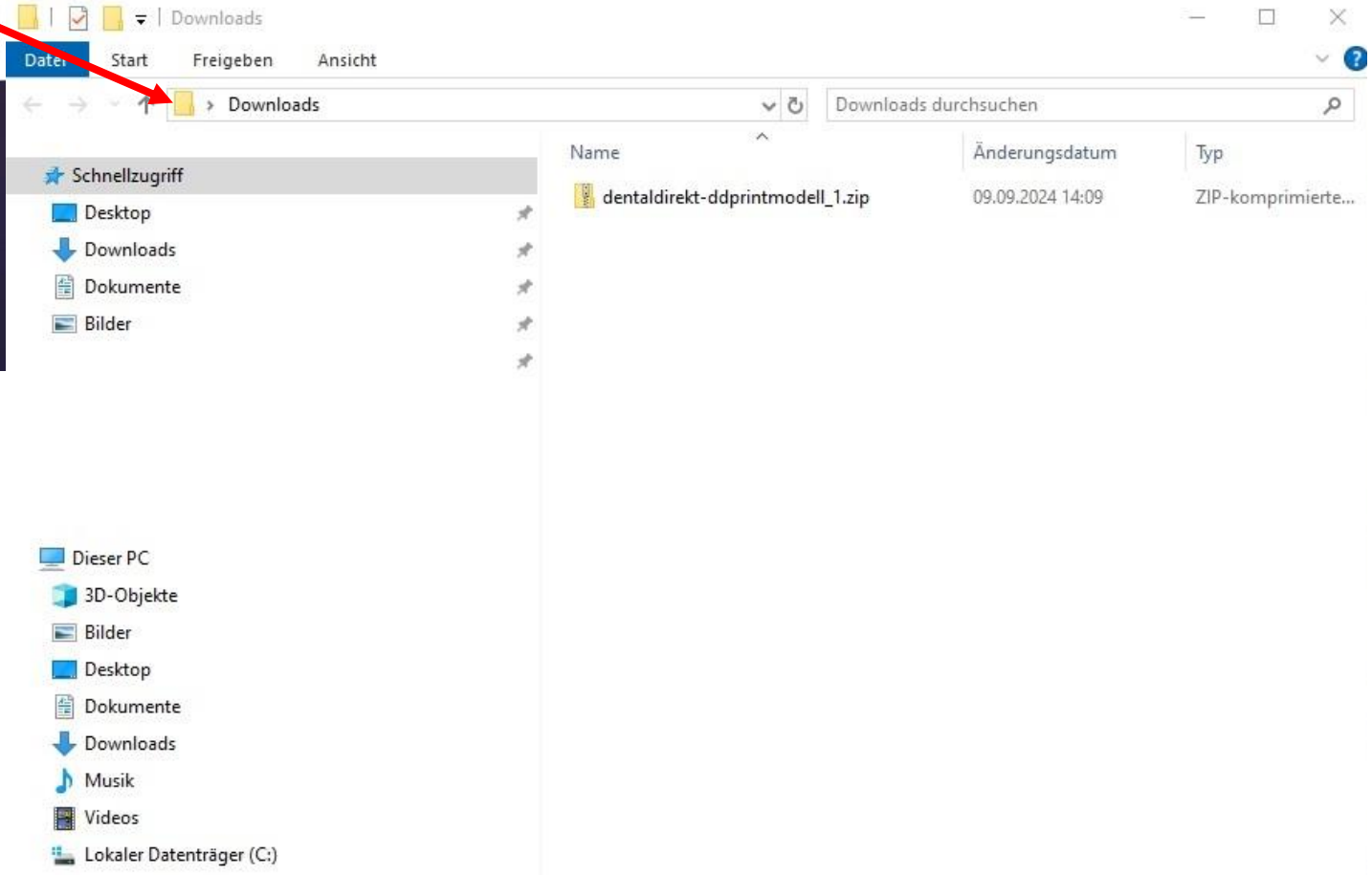
exocad

presents

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Downloads

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Downloads

Downloads durchsuchen

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
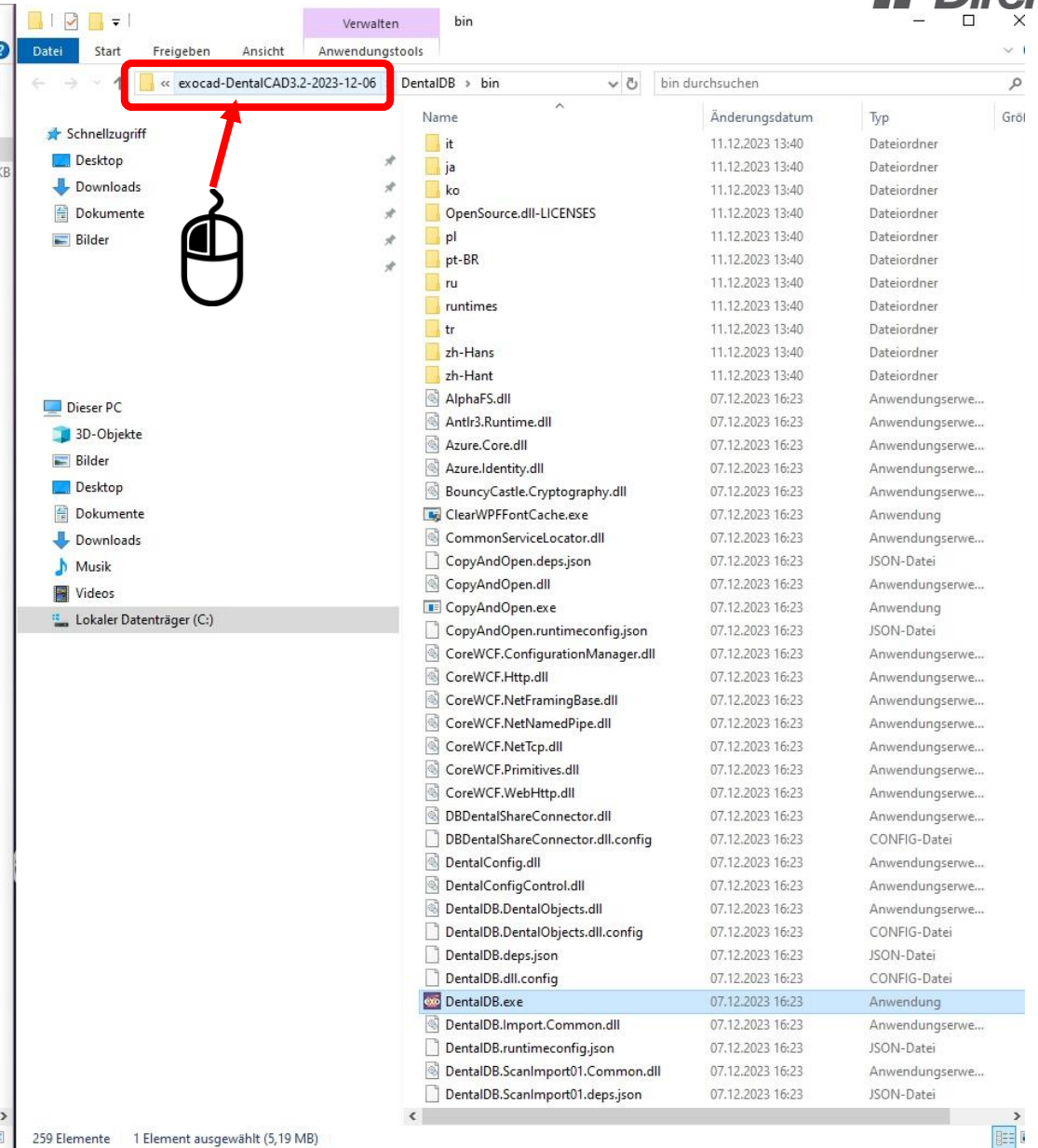
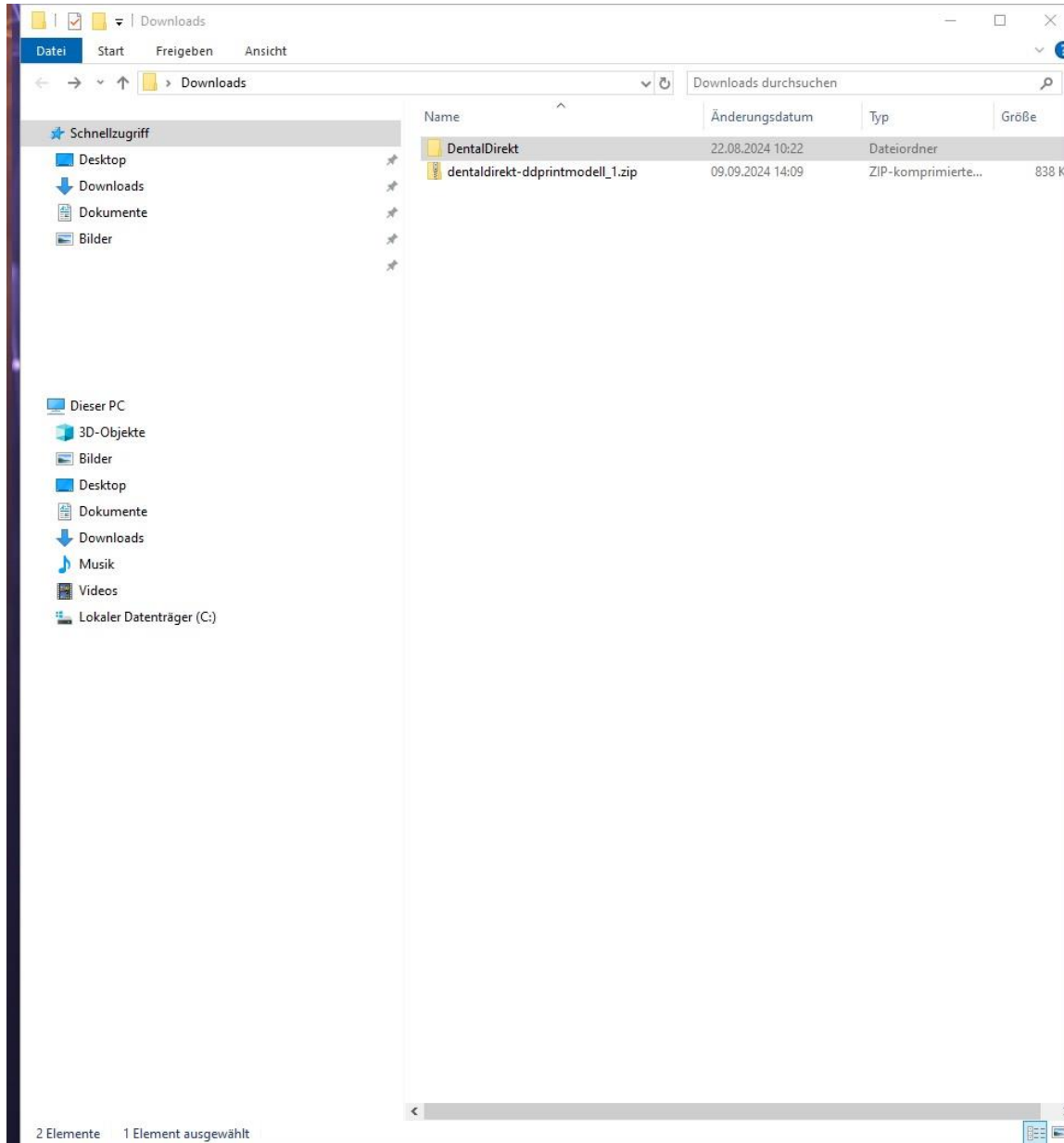
Den Ordner DentalCADApp öffnen


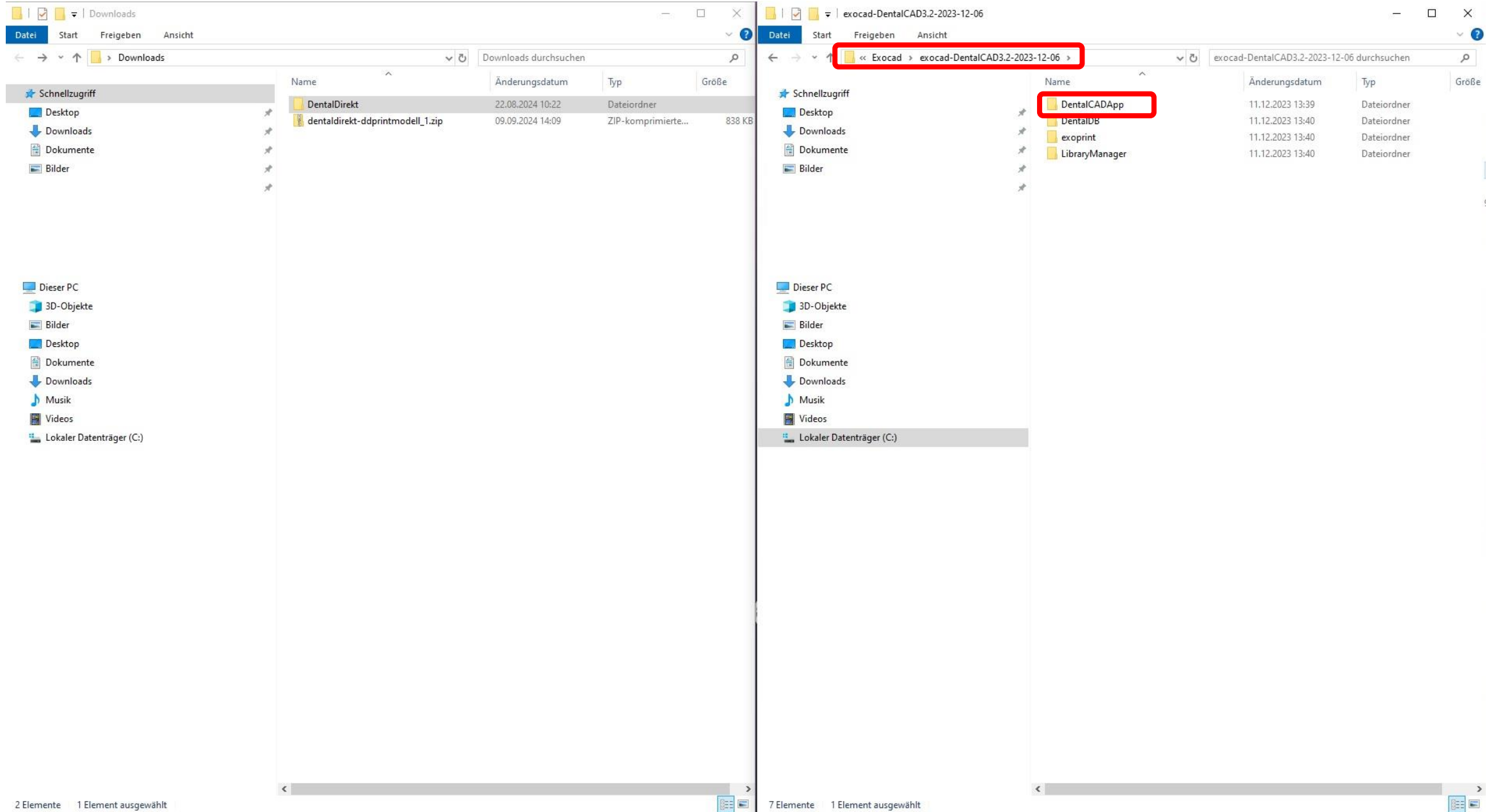
The image shows two side-by-side screenshots of a Windows File Explorer window. The left window shows the 'Downloads' folder with two items: 'DentalDirekt' and 'dentaldirekt-ddprintmodell_1.zip'. The right window shows the path 'Exocad > exocad-DentalCAD3.2-2023-12-06' selected in the address bar, and the 'DentalCADApp' folder is highlighted in the main pane. The status bar at the bottom of the right window indicates '7 Elemente' and '1 Element ausgewählt'.

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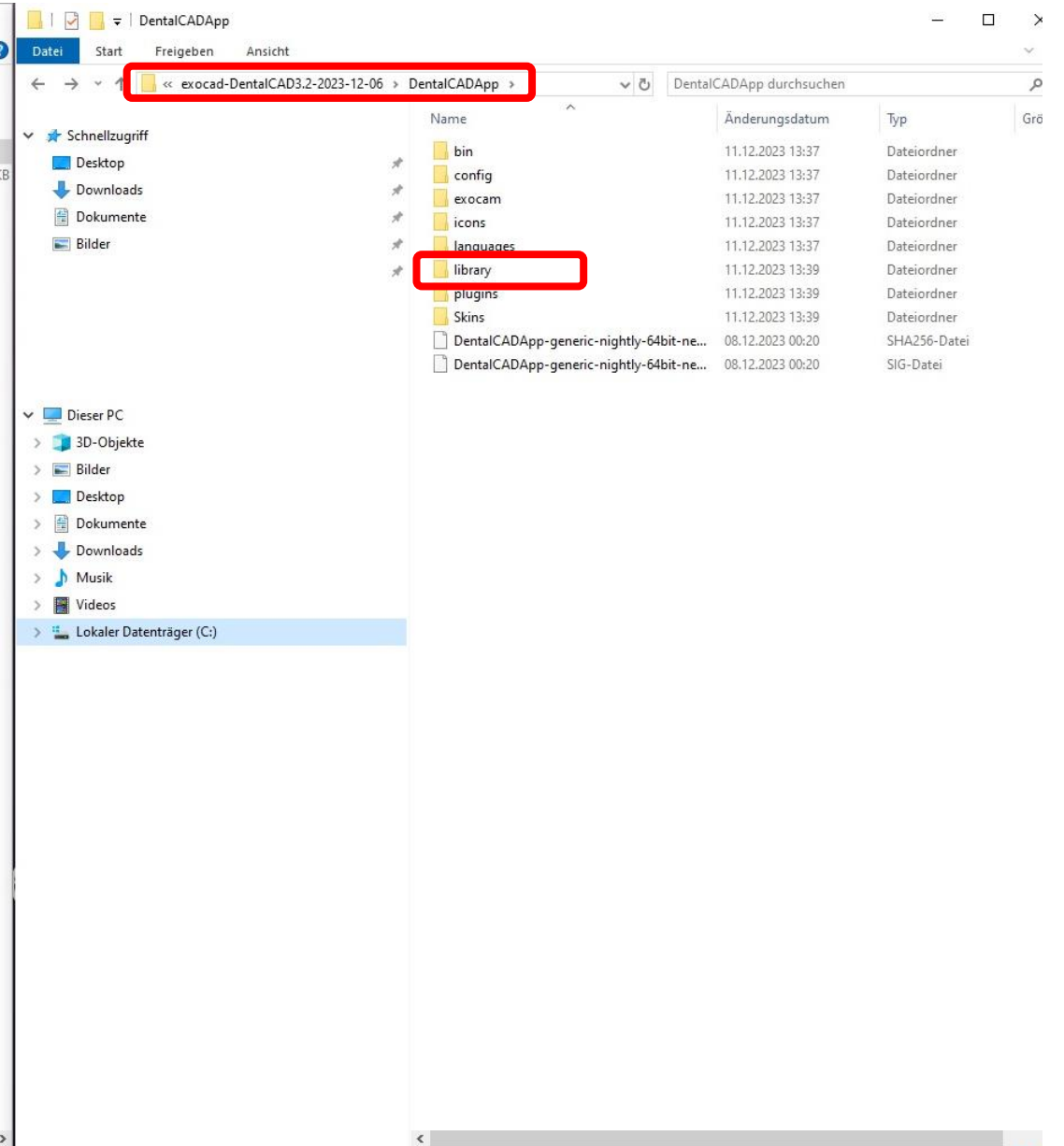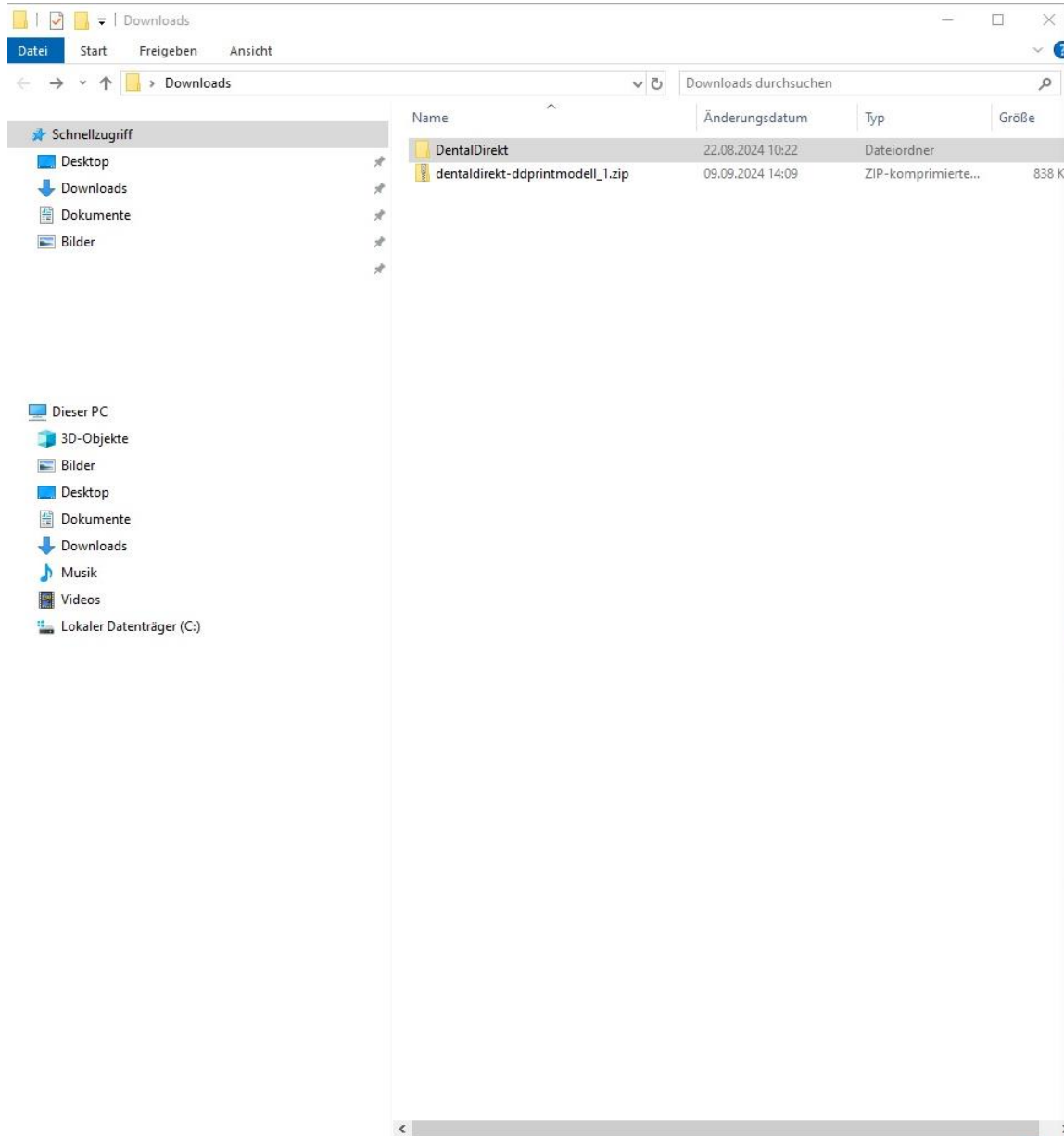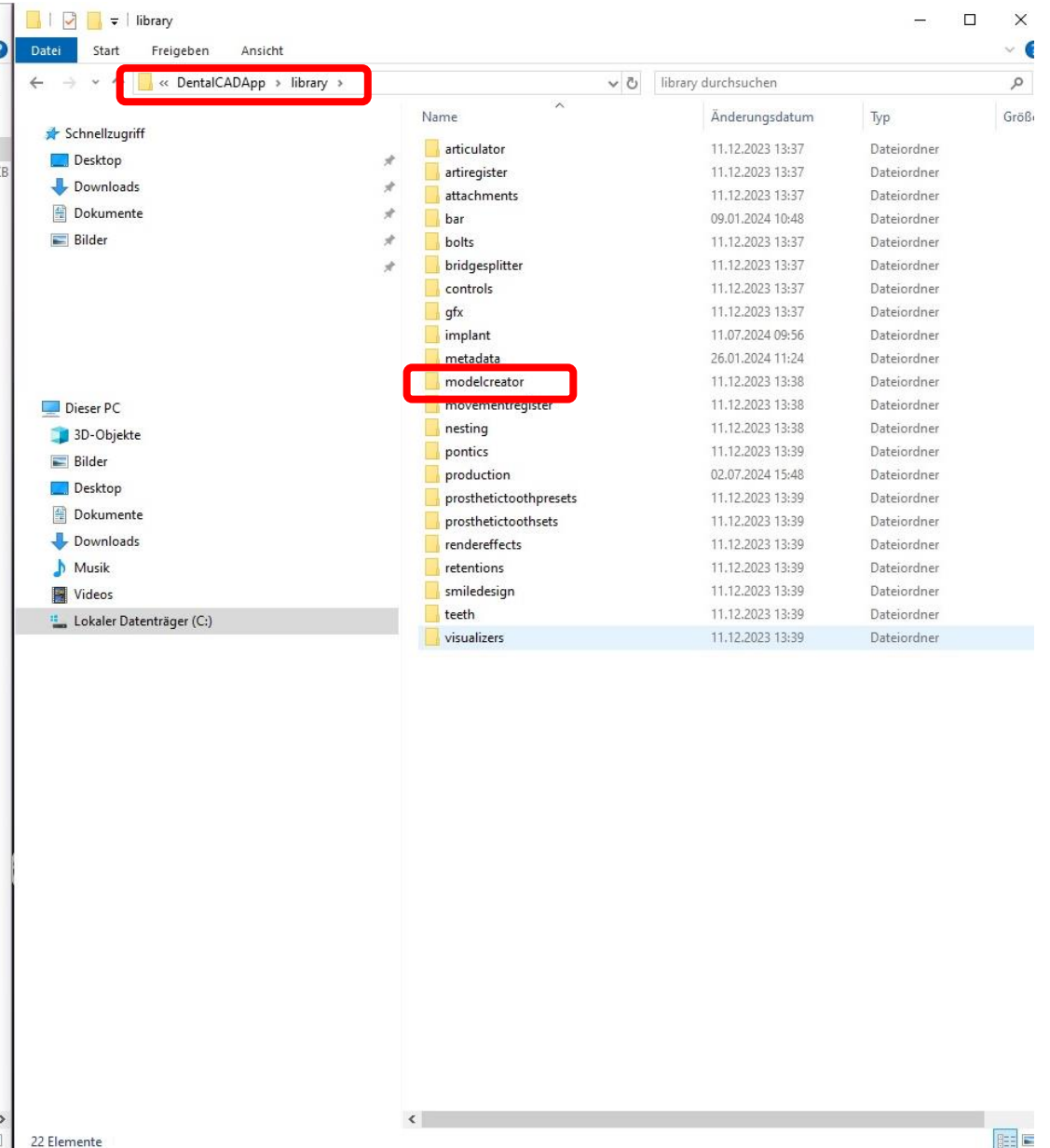
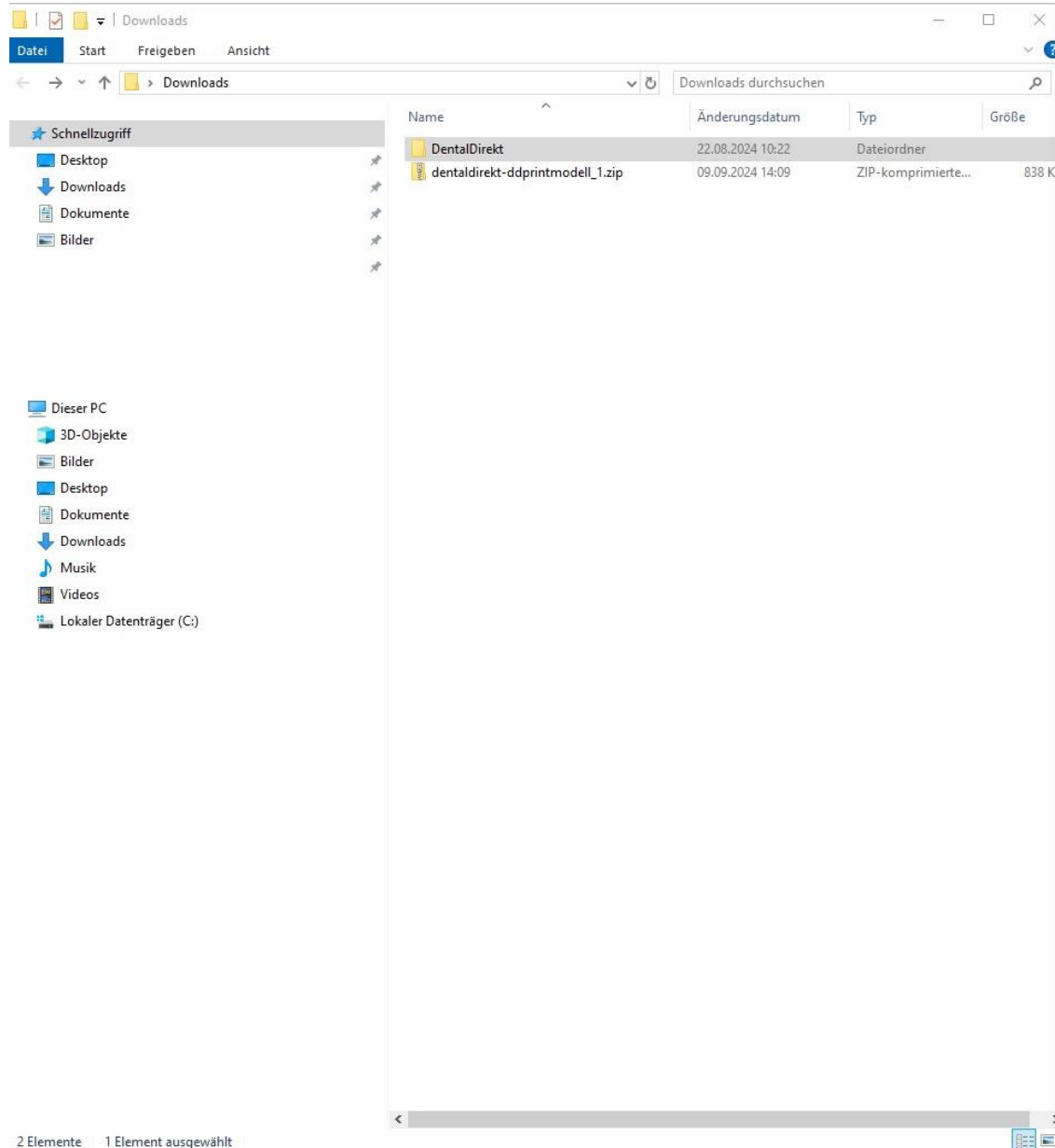
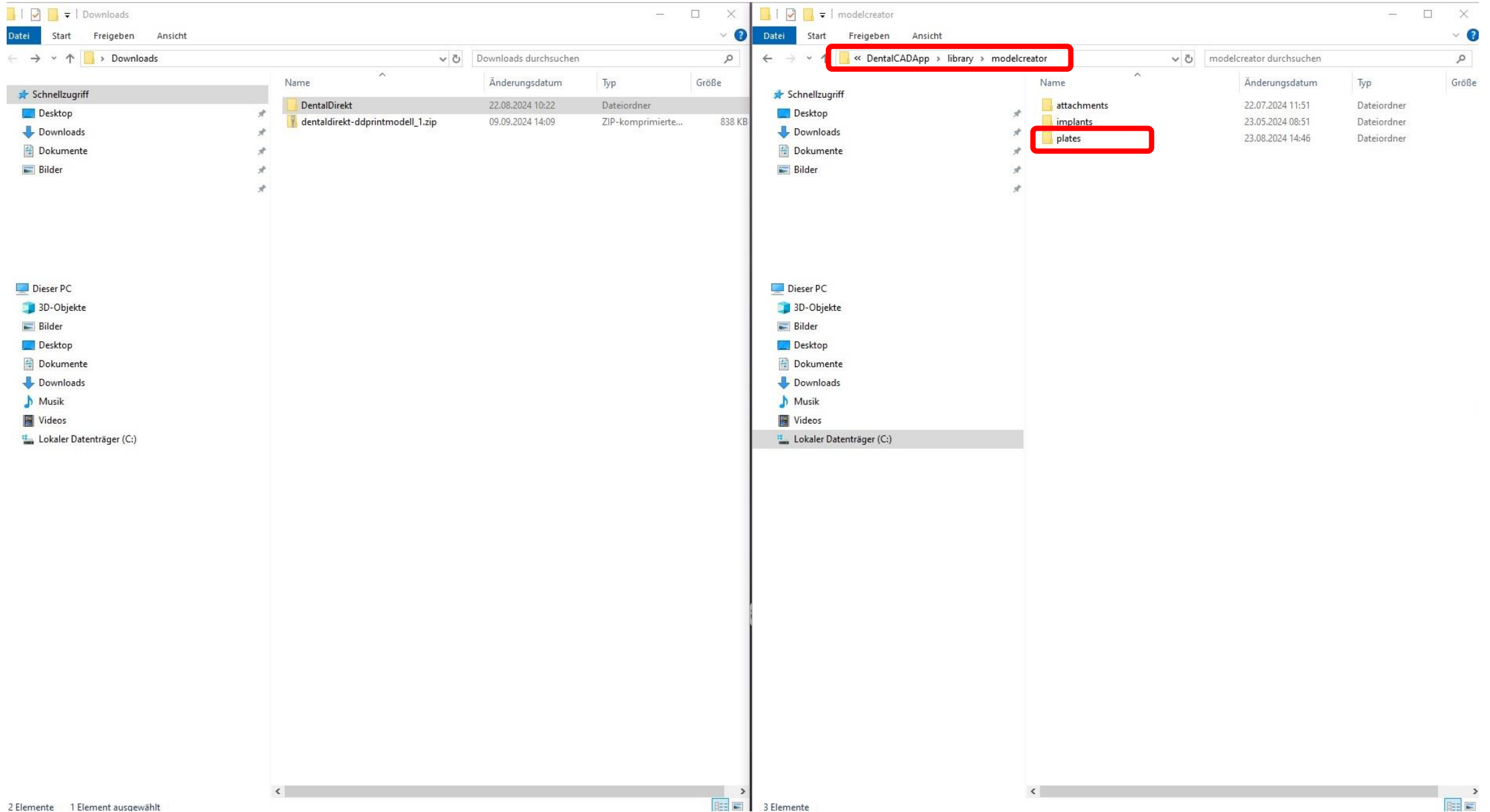
The image shows two side-by-side Windows File Explorer windows. The left window is titled 'Downloads' and shows a folder named 'DentalDirekt' selected. A red box highlights the 'Downloads' folder in the address bar. The right window is titled 'plates' and shows the path '« DentalCADApp » > library > modelcreator > plates'. A red box highlights this path in the address bar. A red arrow points from the 'DentalDirekt' folder in the left window to the 'DentalDirekt' folder in the right window. The word 'kopieren' is written in the center of the image. The status bar at the bottom of the left window shows '2 Elemente' and '1 Element ausgewählt'. The status bar at the bottom of the right window shows '4 Elemente' and '1 Element ausgewählt'.

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i

10:09

The scan data was automatically aligned. To undo the movement, click 'Reset'.

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